HEINE iC1

quick start guide

Subscribe for the HEINE DERManager and download the DERManager app in the AppStore.

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Place the iPhone* in the appropriate mobile phone case until it engages. Please make sure the camera is in the correct position.

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Attach the mobile phone case to the HEINE iC1 until it engages.

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Press your thumb through the opening of the mobile phone case to disengage the iPhone*.

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Charging the HEINE iC1 with the supplied USB cord.

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*Apple, the Apple logo and iPhone are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc.

Cleaning and disinfection

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Please note: to prevent damage to the device, do not spray disinfectants or detergents directly onto the contact plate.

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Recommended cleaning agent for dermatoscope: Neodisher® MediClean

Recommended disinfectant agent for dermatoscope: quaternary ammonium compounds (e.g. Cleanisept® wipes or Microbac® tissues)

Recommended cleaning agent for contact plate: Neodisher® MediClean

Recommended disinfectant agent for contact plate: quaternary ammonium compounds (e.g. Microbac® tissues)

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We reserve the right to change specification without notice.

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