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HEINE EasyClean LED Laryngoscope Handles

Please read and follow these instructions for use of and keep them for future reference.

Intended Use

The internally powered HEINE laryngoscope handle is intended for the transient use with laryngoscope blades according to the "green standard" ISO 7376 for tracheal intubation. It should only be operated by qualified medical personnel within a professional healthcare facility or emergency medical services environment.

For U.S. only:

Federal law restricts this device to sale by or on the order of a Physician or Practitioner!

Warnings and Safety Information

CAUTION! This symbol indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

NOTE! This symbol indicates valuable advice. This notes are important, but no related to hazardous situations.

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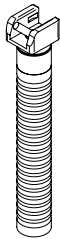
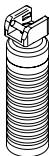
Setting up

EasyClean LED / EasyClean LED SLIM

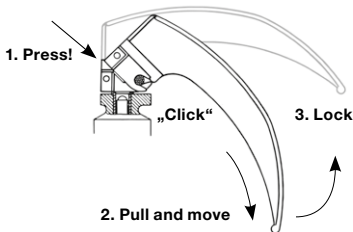
Insert the batteries (b) into the handle (a) leading with the positive terminal. Screw the bottom insert (c) into handle (a).

EasyClean LED SHORT

Insert the batteries (b), regarding the markings on the plastic, into the casing (c). Insert the casing with the contact first into the handle (a) and screw on the bottom insert (d).

Operation

Fix the blade into the working position (see fig.).

Before starting a clinical application, the state of the internal electrical power source must be tested by turning on the light source.

⚠ Only use laryngoscope blades which comply with the "green standard" ISO 7376 in conjunction with HEINE Laryngoscope handles.

The HEINE laryngoscope handle corresponds to the "green standard" according to ISO 7376. If non-HEINE laryngoscope blades are used, component compatibility and functionality are to be checked before each use!

Only trained personnel are allowed to use a laryngoscope for intubation purposes.

HEINE laryngoscopes are designed for use < 5 minutes.

Check the correct operation of the device before each use! Do not use the device if you notice any moisture in the device or damage to the device. (e.g. rough surfaces, edges, damaged O-rings etc).

Before each use, check the condition of the connector for damages, wear and tear. If defects exist, do not use the laryngoscope handle and replace it with one that is in working order.

We strongly recommend a functional backup of laryngoscopes in an emergency situation.

Hygienic reprocessing

The instruction is available:

- at www.heine.com
- in a paper version which you can request from the address listed

Maintenance

Routinely inspect the condition of the batteries and replace them if necessary.

Changing the batteries

Unscrew the bottom insert from the handle and replace the batteries.

In order to ensure a secure electrical connection, use only batteries from renowned manufacturers:

EasyClean LED: LR14 (Size C)

EasyClean LED SLIM: LR6 (Size AA)

EasyClean LED SHORT: LR03 (Size AAA)

Check the correct function of the device after each battery change.

Service

The device has no components serviceable by the end-user.

General Notes

 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

The expected life cycle amounts with designated use and the observation of warning and safety information as well as the maintenance instructions up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Note to the user and/or patient:

All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority.

If you don't use the device for a longer period of time please remove the batteries in advance.

General Warnings

 Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anaesthetic environments).

This product is not allowed to be entered or used in strong magnetic fields like MRI scanners.

Do not shine directly in the eyes to avoid dazzle, especially if the pupils are dilated. Do not use the laryngoscope for eye reflex testing.

Do not use the device for transillumination of tissue.

Disposal

Perform a hygienic reprocessing before disposal.

 The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

The appendix contains following tables

- Electromagnetic disturbances – Requirements and tests
- Technical specification
- Explanation of the used symbols

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Electromagnetic disturbances – Requirements and tests

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environments.

Statement for the operational environments	Inside hospitals except for: near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances are high.
Performance features of the ME system that have been determined to be essential to the performance	None
Warning	Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
	Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
	Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the laryngoscope handle, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
A list of all cables, transducers and other accessories that are relevant for the EMC compliance	As applicable, see attachment
Test	Compliance
RF emissions CISPR11	Group 1 Class B
Harmonic Emissions	Not applicable
Voltage Fluctuations/Flicker	Not applicable

Technical specification

Environmental conditions for operation	+10 °C to +35 °C 30 % to 80 % rel. humidity 700 hPa to 1060 hPa
Environmental conditions for storage (without batteries)	-40 °C to +70 °C up to 95 % rel. humidity 500 hPa to 1060 hPa
Environmental conditions for transport (without batteries)	-40 °C to +70 °C up to 95 % rel. humidity 500 hPa to 1060 hPa
Nominal voltage	EasyClean LED: 2 x Battery 1.5 V (Size C / LR14) EasyClean LED SLIM: 2 x Battery 1.5 V (Size AA / LR6) EasyClean LED SHORT: 3 x Battery 1.5 V (Size AAA / LR03)
Nominal current	max. 260 mA
Class	Internally powered
Classification according to IEC 62471	Group 2
Protection class	IP68

Erläuterung der verwendeten Symbole**Explanation of utilized symbols****Explication des symboles utilisés****Explicación de los símbolos utilizados****Spiegazione dei simboli utilizzati****Förklaring av symboler som används****Verklaring van de gebruikte symbolen****Forklaring af de anvendte symboler****Symbolforklaring****Käytettyjen symbolien selitys****Explicação dos símbolos utilizados**

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