## HEINE OMEGA 500 with DV1

# HEINE OMEGA 500 with DV1

DEUTSCH	3
ENGLISH	5
FRANÇAIS	7
ESPAÑOL	9
ITALIANO	11
SVENSKA	13
NEDERLANDS	15
DANSK	17
SUOMI	19
PORTUGUÊS	21

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# HEINE OMEGA 500 with DV1

These instructions for use apply to the HEINE DV1 digital camera in combination with the HEINE OMEGA 500 indirect ophthalmoscope.

Please read and follow these instructions for use of and keep them for future reference.

## Intended Use

The optional DV1 is a digital camera that provides via a USB 2.0 interface contactless high resolution digital images and a fast live stream. It is used as an educational tool and documentation to show the examiner view to a larger audience or for recording medical techniques for teaching. The DV1 presents no images for diagnosis, pathological analysis, operation or classification of health or disease.

### ▲ For U.S. only:

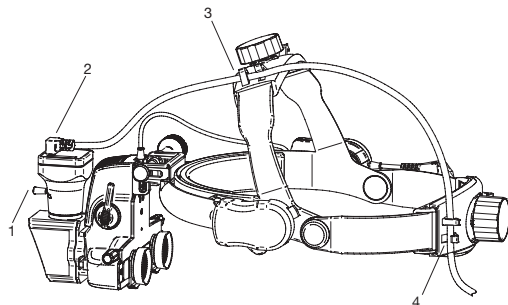
**Federal law restricts this device to sale by or on the order of a Physician or Practitioner!**

## Warnings and Safety Information

**▲ Caution!** Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow, foreground color black.)

**● Note!** Note indicates valuable advice in terms of installation, operation, maintenance or repair. Notes are important, but not related to hazardous situations.

## Product overview

- 1 Focus Lever
- 2 USB port (mini-USB)
- 3 Mounting for USB cable to the headband
- 4 Fastening USB cable to the back of the head band (only for use with mPack)

## Setting up

**▲** Only connect devices to the camera, which correspond at least to the standard of IEC 60950-1 Information technology devices. And as well have a UL listing or a GS mark.

### Technical requirements

To ensure a smooth data flow for the video recording the connected computers must meet the following minimum requirements:

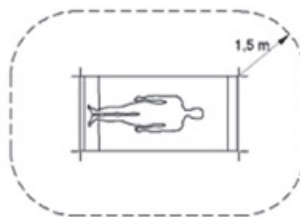
- Processor: Intel Dual Core 2.2 GHz
- Working memory: 2 GB
- Graphics card: 24 bit
- Operating system: Windows XP / 7 / 8 / 10
- Monitor: Resolution of 1280 x 1024 pixels or higher, refresh rate of at least 60 hertz, colour depth, True Color (32 bit) colour calibrated
- DirectX 9.0c or higher
- USB 2.0 port (when using a footswitch another USB port is required)

### Software Installation

Before first use, the installation of the software and the camera driver is necessary. Please follow the installation instructions that come in the support area (see section „OMEGA 500 with DV1“) of the website <http://dv1.heine.com>

## Assembly

**▲** The following steps must be performed outside the patient environment.

Plug the USB cable into the mini USB port of the camera. Then connect the USB cable to the overhead band of the headset (3). If you use the mPack as the power source for OMEGA 500, please also attach the USB cable to the rear head band (4). Fit the OMEGA 500 to the DV1 camera and adjust the head band so it sits securely and comfortably. When doing so, ensure that you have a horizontal view into the eyepiece of the OMEGA 500 with DV1, in particular if you are looking at the ophthalmoscope loupe. Further information on the operation and settings of the OMEGA 500 indirect ophthalmoscope can be found in the instructions for use of this device.

## Operation

**●** The following steps must be performed or checked every time the computer is restarted outside of the patient environment.

1. Start the computer
2. Connect the instrument to the power supply. Check the function by briefly turning on the device.
3. Connect the USB cable.
4. Open the programme IC Capture on the PC, preferably by clicking on the corresponding configuration file.
5. The DV1 digital camera is automatically detected by the computer. Confirm your selection with „OK“.
6. If IC Capture was not started with the configuration file, open the appropriate configuration file.
7. Set video compression.
8. Put on the instrument and adjust headband for a comfortable fit.
9. Adjust focus using the focus selector switch (1). Then look at an object that is approximately in working distance. Possibly a second person is helpful for this setting.

A detailed description of the video and photo recording incl. the specified settings can be found in the installation instructions.

### Video recording

1. Show toolbar or recording window
2. Start recording by clicking the record button.
3. Stop recording with stop button. The data are stored automatically in the predefined directory.

### Photo Shoot

1. Activate the video live window by clicking
2. Press spacebar to store the photo in the default folder.

## Hygienic Reprocessing

Instructions on hygienic reprocessing must be adhered to, based on national standards, laws and guidelines.

Classification according to KRINKO: non-critical  
Spaulding Classification USA: non-critical

**▲** Allow the device to cool down before reprocessing.

Before cleaning, disconnect the device from the power sources.

In the event of suspected contamination, carry out hygienic preparation of the instrument.

The device and its accessories are not sterile. Avoid touching with sterile hands, areas or instruments.

The described cleaning and disinfection measures do not replace the specific rules applicable for the establishment.

HEINE Optotechnik only approves the resources and procedures named in these instructions for use.

Cleaning and disinfection may only be carried out by personnel with sufficient hygienic knowledge.

Observe the instructions of the manufacturer of the reprocessing media.

Do not use any spray disinfectant, or any dripping-wet or heavily-foaming cloths. Do not prepare mechanically.

## Procedure

Clean and disinfect the DV1 manually (wipe clean and wipe disinfect).

### Recommended agents:

Cleaning agent: Neodisher® MediClean

Disinfection agent: quaternary ammonia compounds (e.g. Microbac® Tissues)

Conditioning residues on optics may be removed using a low-bacterial, dry, micro-fibre cloth.

Please follow the cleaning notes for the OMEGA 500 in the identically named instructions for use.

## Maintenance and Service

The instruments do not require maintenance or service.

## General Warnings

-  Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.
- Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anesthetic environments)
- Do not modify the device.
- Use only original HEINE parts, spare parts, accessories and power sources.
- Repairs shall only be carried out by qualified persons.
- Route the supply cable in such a way that nobody can trip over it.

## General Notes

-  The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit [www.heine.com](http://www.heine.com).
- The expected life cycle, when the device is normal used and the warning and safety information as well as the maintenance instructions are observed, is up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.
- Store and use the device in dry and dust-free environments only!

## Disposal

-  The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

## Electromagnetic Compatibility

Medical electric devices are subject to special precautionary measures with regard to electromagnetic compatibility (EMC). Portable and mobile high frequency communication equipment can affect medical electric devices.

-  This is a device in the domestic environment, this device may cause radio interference, so that it may be necessary in this case, to take appropriate remedial measures, as e.g. orientation, new arrangement or shielding of the device or restrict the connection to the site.
- The use of accessories, converters or cables other than the ones specified by HEINE might lead to increased emission reduced electrical immunity of the medical equipment.
- The device may not be stacked directly near or used directly beside other devices. If the device is to be operated in a stack or with other devices, the device should be watched to ensure it operates properly in this location.

The appendix contains following tables:

- Guidance and Manufacturer's declaration - Electromagnetic immunity
- Technical specification
- Explanation of the used symbols

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Guidance and manufacturer's declaration – electromagnetic emissions		
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environment.		
Emission test	Compliance	Electromagnetic environment – Guidelines
RF emissions CISPR11	Group 1	The device uses RF energy only for its internal function. Therefore, RF-emission is very low and it is unlikely that any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic Emissions IEC 61000-3-2	Class A	Symmetrical three-phase devices and other devices.
Voltage Fluctuations/ Flicker Emissions IEC 61000-3-3	Passed	

Guidance and manufacturer's declaration - Electromagnetic immunity			
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – Guideline
Electrostatic discharge (ESD) acc. to IEC 61000-4-2	± 6 kV contact discharge ± 8 kV air discharge	± 6 kV contact discharge ± 8 kV air discharge	Floors should be wood, concrete or covered with ceramic tiles. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for mains cables ± 1 kV for input and output lines	± 2 kV for mains cables ± 1 kV for input and output lines	The supply voltage quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV voltage phase – phase, ± 2 kV voltage phase – earth	± 1 kV voltage phase – phase ± 2 kV voltage phase – earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	< 5% U <sub>T</sub> , (>95% dip in U <sub>T</sub> ) for 1/2 period 40% U <sub>T</sub> , (60% dip in U <sub>T</sub> ) for 5 periods 70% U <sub>T</sub> , (30% dip in U <sub>T</sub> ) for 25 periods <5% U <sub>T</sub> , (>95% dip in U <sub>T</sub> ) for 5 seconds	< 5% U <sub>T</sub> , (>95% dip in U <sub>T</sub> ) for 1/2 period 40% U <sub>T</sub> , (60% dip in U <sub>T</sub> ) for 5 periods 70% U <sub>T</sub> , (30% dip in U <sub>T</sub> ) for 25 periods <5% U <sub>T</sub> , (>95% dip in U <sub>T</sub> ) for 5 seconds	Mains power quality should be that of a typical commercial or hospital environment. If the user of the device requires continued operation during power mains interruptions, it is recommended that the device be powered by a UPS (uninterruptible power supply) or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Comment: U <sub>T</sub> is the a.c. supply voltage prior to application of the test level.			

Guidance and manufacturer's declaration – electromagnetic immunity			
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment Guidelines
Conducted RF IEC 61000-4-6	3 V <sub>eff</sub> 150 kHz to 80 MHz	3 V <sub>eff</sub>	Portable and mobile RF communication equipment should be used no closer to any part of the device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.
Radiated HF IEC 61000-4-3	3 V/m 80MHz to 2,5GHz	3 V/m	<p><b>Recommended separation distance:</b>  <math>d = 3,5/3 \cdot \sqrt{\text{P/W}}</math>  <math>d = 3,5/3 \cdot \sqrt{\text{P/W}}</math> 80 MHz to 800 MHz  <math>d = 7/3 \cdot \sqrt{\text{P/W}}</math> 800 MHz to 2,5 GHz</p> <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).            Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey<sup>a</sup>, should be less than the compliance level in each frequency range.<sup>b</sup>            Interference may occur in the vicinity of equipment marked with the following symbol:</p> 

Note 1: At 80Hz and 800MHz, the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a Field strengths from fixed transmitters, such as base stations for radio (cellular / cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy.  
To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the device is used exceeds the applicable RF compliance level above, the device should be observed to verify normal operation.

If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the device.

b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V / m.

**Recommended separation distances for portable and mobile RF communication equipment and the device**

The device is intended for use in the electromagnetic environment in which radiated RF disturbances are controlled. The customer or user of the device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communication equipment (transmitters) and the device as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)		
	150 kHz to 80 MHz $d = 3,5/3 \cdot \sqrt{P}$	80 MHz to 800 MHz $d = 3,5/3 \cdot \sqrt{P}$	800 MHz to 2,5 GHz $d = 7/3 \cdot \sqrt{P}$
0.01	0.1	0.1	0.2
0.1	0.4	0.4	0.7
1	1.2	1.2	2.3
10	3.7	3.7	7.4
100	11.7	11.7	23.3

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be determined using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note 1: At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

**Technical specification**

Environmental conditions for operation	+10 °C to +35 °C 30 % to 75 % rel. humidity 700 hPa to 1060 hPa
Environmental conditions for storage	+10 °C to +55 °C 10 % to 95 % rel. humidity 500 hPa to 1060 hPa
Environmental conditions for transport	-40 °C to +70 °C 10 % to 95 % rel. humidity 500 hPa to 1060 hPa
Protection class	II
Resolution camera	2592 x 1944 pixels (5 megapixels)
Format	1/2,5" CMOS
Interfaces	mini-USB (camera), USB 2.0 (PC)
Power supply	Via USB 2.0 (5 V) – cable length 3 m
Weight	580 g

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**Explanation of utilized symbols**

The following symbols are used on the device or on the packaging:

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